

## Honeywell Life Care Solutions Glucose Instructions

---

### LifeScan One Touch® Ultra®2



### **\*\*Take your glucose readings BEFORE taking your vital signs\*\***

1. Verify that you have the correct glucose meter and strips.
2. Insert a NEW test strip with the bars facing you into the glucose meter.
3. Obtain a blood sample.
4. Put blood sample on test strip.
5. Remove the strip and throw away.

### **Using the Glucose Meter with the Monitor**

1. Make sure the test strip is out of the meter and the meter is turned off.
2. Wait for the monitor to prompt you to attach the glucose meter to the cable.
3. Attach the cable to the glucose meter.
4. When the monitor says "data upload is finished" remove the glucose meter from the cable.